GRAPHIC SYMBOLS USED IN LABEL			
Symbol	Description	Symbol	Description
	Do not re-use – Single use		Use by date
	Batch code		Sterilized using ethylene oxide
	Catalogue number		Consult instructions for use
	Do not use if package is damaged		Manufacturer
	Do not re-sterilize		EC certification according MDD 93/42
	Non-pyrogenic		Date of manufacture

AMS Group S.r.l. denies all responsibility for any damage caused to the patient and/or medical staff if the system is used anomalously, improperly or due to mistaken movements and unqualified staff.

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10. The body of the **Resaloon** catheter is provided with a white marker, that indicates:

- **Correspondence Blue-White:** alignment of the tip of the balloon catheter with the distal tip of the video-guided catheter Resascope. Do not inflate.
- **Correspondence Back line:** portion of the balloon set to half compared to the tip of the video-guided catheter Resascope. Do not inflate.
- **Correspondence White-Blue:** complete out of the balloon respect the tip of the video-guided catheter Resascope. It is possible to inflate.

11. When the white stretch is entirely inside the proximal valve of the Resascope and the images transmitted by the optical fiber identify an adequate inflation and positioning condition, act on the plunger of the syringe 1ml, previously connected.

Warning: always follow in real time the effectiveness of the balloon dilation through the monitor.

12. Move with care the balloon catheter when is inflated. The balloon catheter is designed to penetrate into the epidural adhesions and perform the dilation. Advance or withdraw **Resaloon** catheter when inflated. It could cause injuries or fractures of the same that can compromise the correct operation.

Warning: do not move the video-guided catheter Resascope if the balloon catheter is inflated. Otherwise the balloon catheter **Resaloon** could burst and cause injury to the patient.

13. Ensure, before the complete removal of the system, that the balloon catheter **Resaloon** is completely deflated.

Warning: The syringes supplied, 5ml and 1ml, can be filled with contrast liquid if you choose to make the distal position of the balloon visible by fluoroscopy.

INSTRUCTIONS FOR USE

Warning: for the correct use of the balloon catheter **Resaloon** cod. *R-LOON-02* follow the instructions below.

1. Pull out the epidural balloon catheter **Resaloon** cod. *R-LOON-02* and both syringes supplied from the package tray. Remove the balloon catheter protective tube, without removing the compression sheath [orange] from the balloon catheter.
2. Withdraw 3ml of saline in the 5ml syringe for the “Venting” procedure (i.e. the removal of air from the lumen of the balloon catheter).

Warning: The 5ml syringe must be used only for the “Venting” procedure from the catheter cavity. Do not use it to inflate the balloon.

3. Connect the 5ml syringe to the catheter through the Luer Lock and verify that the wing tap is in line with the body of the catheter (open position).
4. Position the syringe vertically, with the plunger upwards. Pull the plunger upwards and then let it go back to its normal position. You will see some air bubbles racing back into the syringe. The “Venting” procedure can be considered completed when no air bubbles appear anymore; then, close the wing tap by positioning it perpendicularly to the body of the catheter and detach the syringe from the Luer Lock.
5. Remove the compression sheath [orange] from the balloon catheter.
6. Completely fill with saline or contrast liquid the 1ml syringe contained in the package tray, eliminating any air bubbles.
7. Connect the 1ml syringe to the proximal valve with Luer Lock connection.
This syringe is equipped with a special membrane, which ensure that the volume of saline solution injected to inflate the balloon (prior to “Venting” procedure) is always constant, allowing the way out of any excess of solution.

Warning: despite the 5ml syringe provided is calibrated, the inflation of the epidural balloon catheter **Resaloon** with more bolus of saline or contrast liquid may cause the outbreak of the balloon with serious consequences for the patient.

8. Introduce the epidural balloon catheter **Resaloon** into the video-guide, using the appropriate back valves “TOOL1” or “TOOL2”.

Warning: it is recommended not inflate the balloon before introducing it into the video-guided catheter Resascope.

9. Open the wing tap (previously closed) by aligning it with the body of the catheter.

Warning: before inflation, make sure through optical vision, that the distal part of the terminal part of the balloon catheter is completely out from the distal tip of the Resascope and is clearly visible on the screen.

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INSTRUCTIONS FOR USE

RESALOON EPIDURAL BALLOON CATHETER

INDICATIONS

The epidural balloon catheter **Resaloon**, cod. *R-LOON-02* is a IESS® surgical accessory, that used with the video-guided catheter Resascope allows to:

- Explore the epidural space and evaluate its anatomical and pathological conditions
- Mechanically remove and detach scar tissue adhesences
- Facilitate the opening of local stenotic occlusions during the targeted investigation of epidural pathological conditions encountered.

CONTRAINDICATIONS

The epidural balloon catheter **Resaloon**, cod. *R-LOON-02* is contraindicated when, at the physician's discretion, its use can threaten the patient's health, due to known allergies to its components.

Warning: spinal anaesthesia is contraindicated for procedures involving minimally invasive surgical approach of the epidural space (spinal endoscopy).

Do not use distilled water solutions!

Possible complications: as in every surgical procedure possible complications may occur during peri-postoperative phase.

Specifications of this procedure may include: headache, retinal or intracranial hemorrhage, dural puncture, meningoencephalitis, discitis.

Possible side effects: possible side effects may include pain at the insertion site and paresthesia/dysesthesia in the perioperative phase.

GENERAL INSTRUCTIONS

The epidural balloon catheter **Resaloon** cod. *R-LOON-02* must be used exclusively with the Resascope video-guided catheter. Its introduction into the epidural space is effected through insertion in the back valve of the Resascope dedicated tools ("TOOL1" or "TOOL2").

Warning: Do not use if package is damaged. Sterile and non-pyrogenic. Single use.

Warning: Prepare the video-guided catheter Resascope following the relevant instructions for use.

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AMS Group S.r.l. s.u.